24

SET TO RESCUE

Installation Manual

Parts-list

This Rotaid package contains:

- Installation manual
- Rotaid 24/7 voucher
- User Guide
- Drilling template
- Polycarbonate backplate
- Polycarbonate cover
- Rotaid 24/7 module
- Transformer unit and power cord
- Fastening materials

Screw Set:

4x Tapping screw

4x Plug

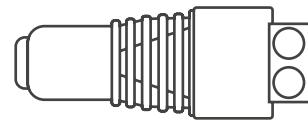
2x Eject screw

1x Bolt


1x Nut

1x Connector

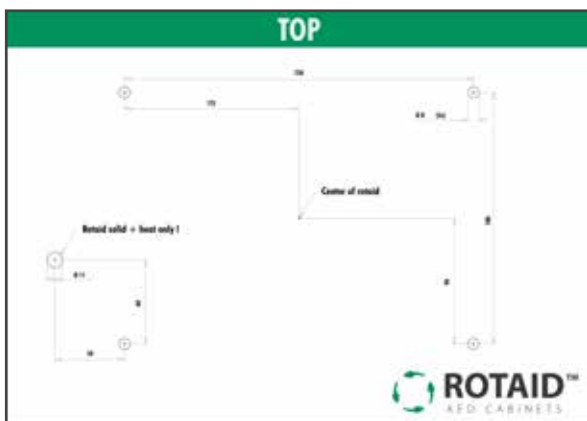Drilling Template:

1x Sleeve

Index

- pag. 4**  English instructions
- pag. 6**  Nederlandse instructies
- pag. 8**  Instructions en Francais
- pag. 10**  Deutsche Anleitung
- pag. 12**  Istruzione Italiano
- pag. 14**  Polskie instrukcje
- pag. 16**  Danske instruktioner
- pag. 18**  Norske instruksjoner
- pag. 20**  Svenska instruktioner
- pag. 22**  Instrucciones en español

Rotaid 24/7 contact information

Keizersveld 52D
5803 AN
Venray
The Netherlands

www.rotaid247.com

support@rotaid247.com

+31 (0)478 769 052

ENGLISH INSTRUCTIONS

WARNING

Only a certified company or electrician should install the cabinet since part of the installation comes with an electrical installation. Your cabinet is not yet ready for use!

Determine the location where you wish to mount the Rotaid. Make sure there are no irregularities present at the desired location.

To ensure the defibrillator stays within its operational temperature range during the whole year, the Rotaid should be located in the shade at all times.

Please take into account the fire safety and possible evacuation regulations.

Step 1. Preparations and drilling

The ideal mounting position for the Rotaid is 150 centimeters (59 inches), measured from the center of the cabinet to the floor.

Use the drilling-template to mark the four holes for the mounting positions. Also mark the hole for the cable to the cabinet.

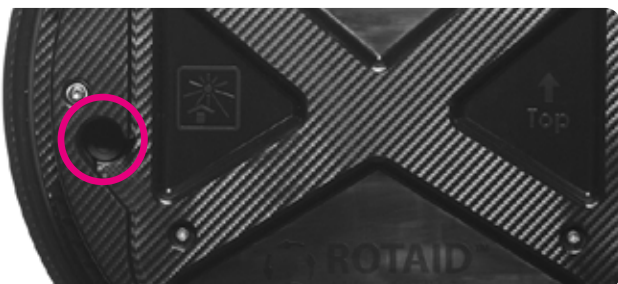
Please drill the four holes using an 8mm drill and a 10mm drill for drilling the cable hole.

Now place the four plastic wall plugs.

Step 2. Cable feed and fixation

Please find assistance from another person to assist you with this step.


On the bottom-left of the backplate a rubber plug is located. Use an awl or screwdriver to puncture this plug to allow for enough space for the power cable.

Run the whole power cable through the hole and wall.

Important! Do not yet attach the white Rotaid 24/7 module to the back plate.

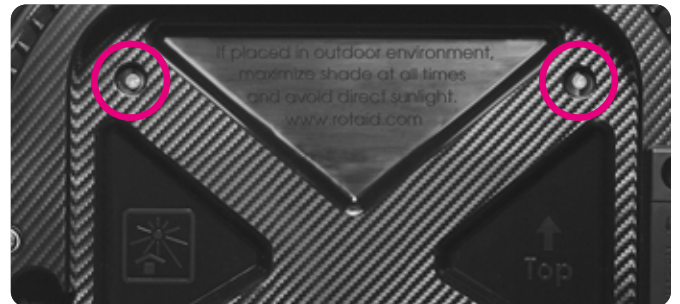Now attach the back plate to the wall by using the four screws.

After this you can mount the 24/7 module via the two small screws. Please use the two holes on the top of the module (see picture).

Step 3. Placement of cover and defibrillator

Attach the nylon strap to the cover using the small bolt and nut.

After mounting the cover you can place the defibrillator on the platform. Use the elastic band to secure the defibrillator.

You can now place the cover on the backplate. Align the recess on the top of the cover with the special marker on the top left of the back plate. Make sure the steel hook on the bottom is in the cover before you start rotating. Turn the cover to the right until it cannot move further to the right. The steel hook now falls through the opening at the bottom of the hood. Your Rotaid is mounted correctly.

Please now proceed with Step 4.

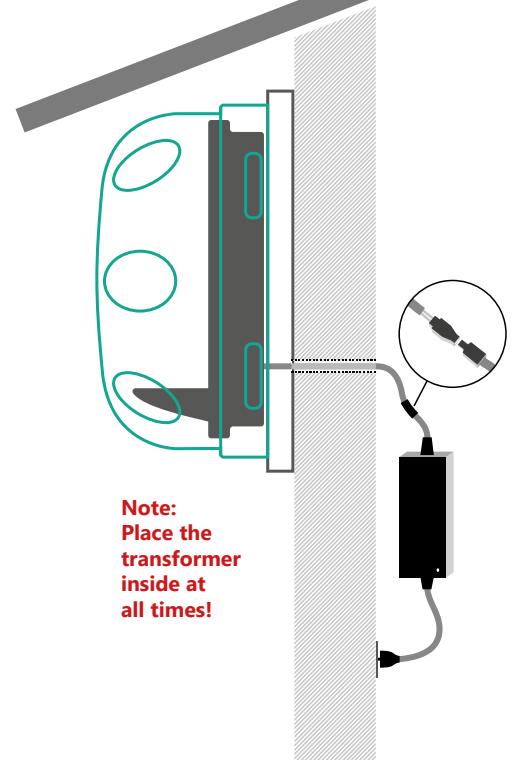
Step 4. Electrical wiring

IMPORTANT!

Always place the transformer unit inside since it is not suitable for outdoor usage. Never place the transformer unit inside the Rotaid cabinet!

Cable brown +
Cable blue -

1. Cut the cable to the required length and strip the outer cable (about 2 cm)
2. Remove 5 mm of the isolation of both cables.
3. Place the crimp sleeve insulator over the cable.
4. Connect both wires to the connector
5. Place the crimp sleeve over the connector and use a heat gun to fixate. You can also isolate / fixate the connector with tape.
6. Connect the connector to the plug of the transformer (male – female)
7. Now connect the power plug to the transformer and plug the power cable into the wall outlet.
8. The LED lighting in the cabinet is now activated and will be on constantly. This indicates your installation has been successful.
9. Your Rotaid 24/7 is now ready for use!

Step 5. Activate your Rotaid 24/7 system

Please follow the instructions on the Rotaid 24/7 voucher to activate your Rotaid 24/7 account.

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